CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2515

CS Prone C-Flex Compatible Cushion Small


CARE SURGICAL

Product Code: CSM2515

CS Prone C-Flex Compatible Cushion Small

Use with the Allen® C-Flex® Head Positioning System to support to your patient's forehead and cheeks. Ensure the eyebrows are visible to avoid direct pressure to the eyes.


⚠ Only to be used by trained clinicians.

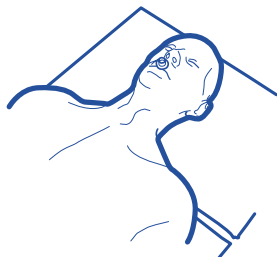

1 With patient supine, disconnect ET tubing

2 Place cushion onto patient's face*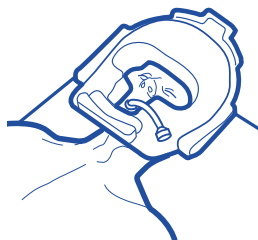

3 Turn the patient to the prone position and locate the face cushion in the C-Flex Head Positioner. Position the ET tubes through the channel as shown

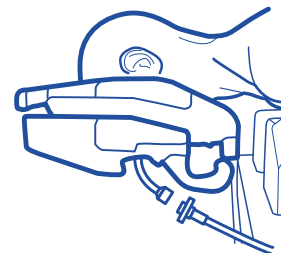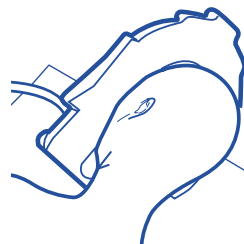4 Ensure the tubing is not trapped between the cushion and the patient's face

Follow the instructions supplied with the Allen C-Flex Head Positioning System to achieve a suitable neutral neckline

*See overleaf for guidance on correct positioning

Allen® is the registered trademark of Allen Medical Systems Inc.

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland


⚠ DO NOT REUSE! If reused injury to patient could occur due to issues with infection control and deterioration of the product.
⚠ SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.
⚠ DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.

5 060624 122484

EU REP

The Forehead & Cheeks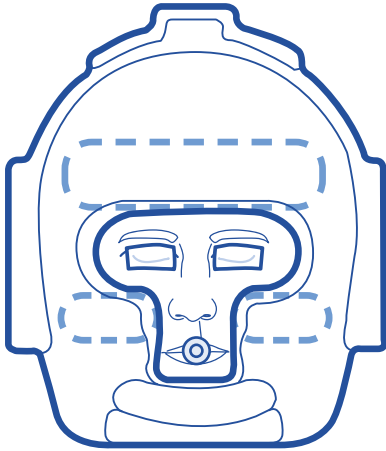

Possible problems

- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland

CARE SURGICAL

Produktkode: CSM2515

CS Prone C-Flex Compatible Cushion Small

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