CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2519

CS ProneOne™ Head Support- Small

CARE SURGICAL

Product Code: CSM2519

CS ProneOne™ Head Support- Small


*Suitable for patients up to 8 years

The CS ProneOne™ offers support to your patient's forehead and cheeks. Ensure the eyebrows are visible to avoid direct pressure to the eyes.


⚠ Only to be used by trained clinicians.


Step 1 Prepare intubated patient in the supine position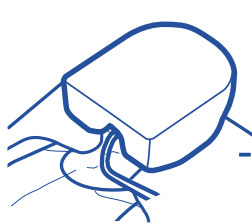

Step 2 Place cushion onto patient's face*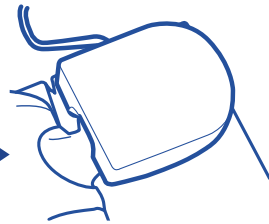


Step 3 Position ET tube to the intubation slot

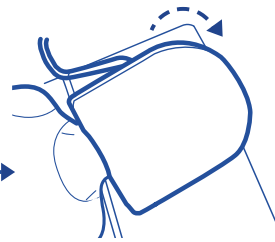Step 4 Turn patient to prone position

Step 5 Ensure tubing is still connected and cushion is in the correct position, not obstructing the patient's eyes or airway

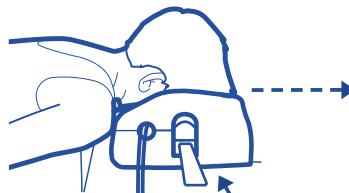Step 6 Slide mirror into place

Step 7 Using the mirror ensure the face is correctly positioned in cushion ensuring tubing is connected and eyebrows are visible

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

⚠ **DO NOT REUSE!** If reused injury to patient could occur due to issues with infection control and deterioration of the product.
⚠ **SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.
⚠ **DO NOT USE** products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.

5 060624 120039

The Forehead & Cheeks

Possible problems


- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement

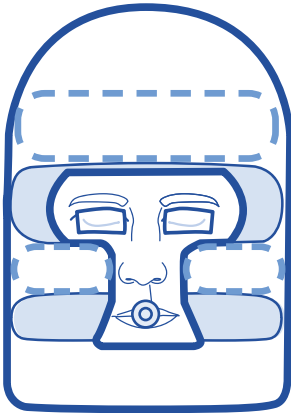See the instructional video
at www.care-surgical.com

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland

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