




# CARE SURGICAL

## CS Prone Arm Secure - Large

Support patients arms securely with the CS Prone Arm Secure.

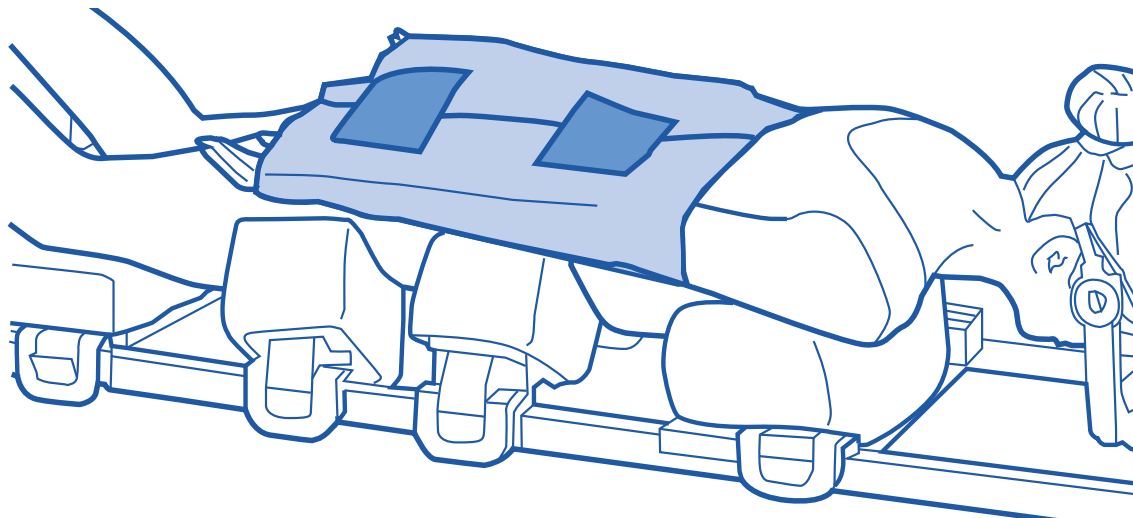

Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

**[www.care-surgical.com](http://www.care-surgical.com)**

Care Surgical Supplies Limited, Office 2, 12a Lower Main  
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity  
Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 122040




LOT Number:


EC REP

Product Code: CSM-2674 Rev. 6 02/24


## Support patients arms securely with the CS Prone Arm Secure.

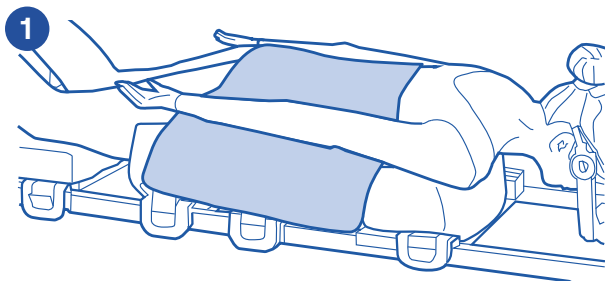


Place the CS Arm Secure under the patients arms in the prone position with the laminated side facing upwards.

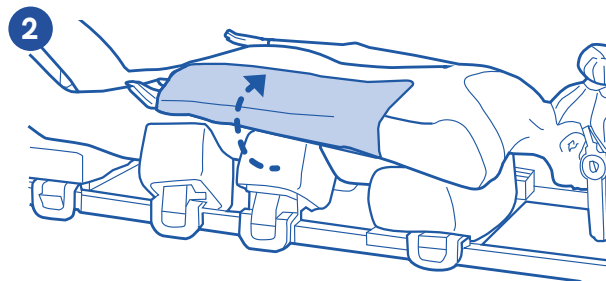

Fold the CS Arm Secure around the patients arm towards the spine.

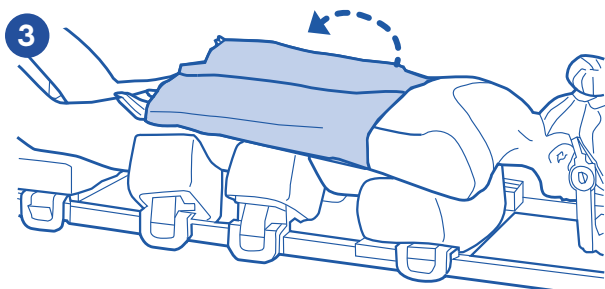

Fold the opposite side of the CS Arm Secure towards the spine.

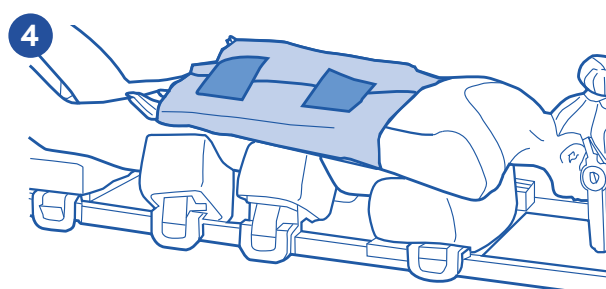

Secure with Velcro®.



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107



Medical Device



UK Conformity  
Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 122040

**www.care-surgical.com**

Care Surgical Supplies Limited, Office 2, 12a Lower Main  
Street, Lucan Co. Dublin, K78 X5P8 Ireland

EC REP

Product Code: CSM-2674 Rev. 6 02/24