





CARE SURGICAL

CS Disposable Leg Holder


CS Disposable Leg Holder - For use with compatible arthroscopic leg holder to provide safe and comfortable support.

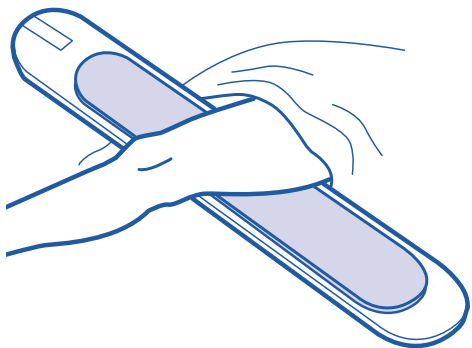

- 1** Position the leg holder pad under the patients leg with the white side facing outwards.

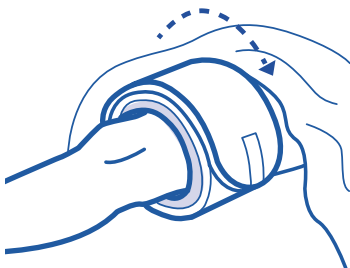

- 2** Wrap the leg holder pad securely around the patients leg and secure using the strap.

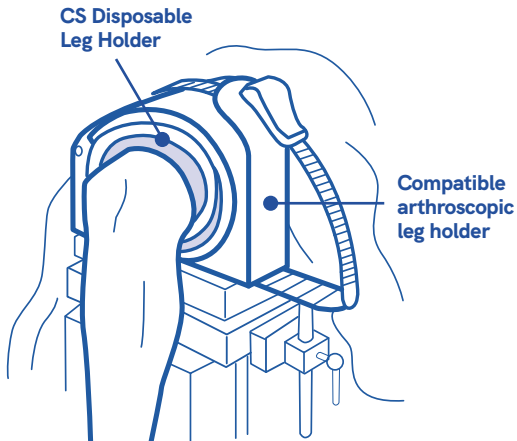

- 3** Use with compatible arthroscopic leg holder.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity
Assessed



Do Not Reuse



CE Mark



Latex Free



Non Sterile



5 060624 127380



LOT Number:

EC REP

Product Code: CSM-2738 Rev 2 02/24