




CARE SURGICAL

CS Leg Support Drape & MF Large (use with CSM-2634)

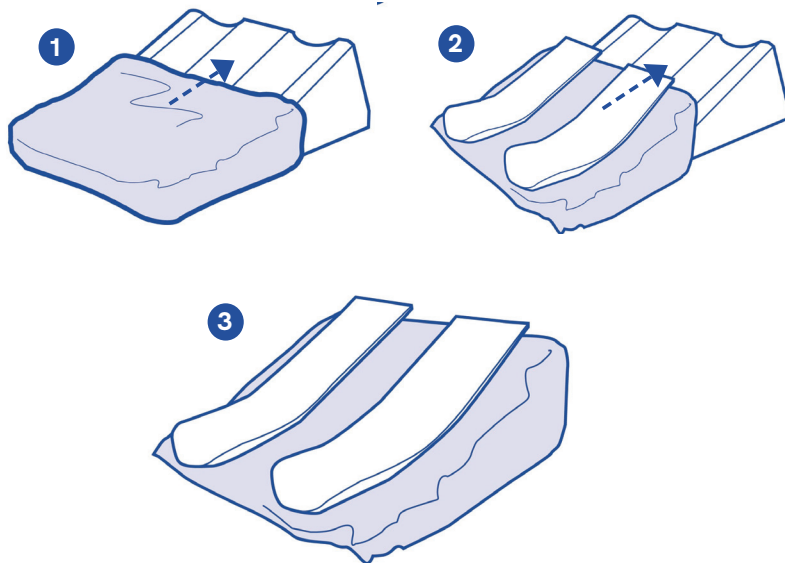

CS Leg Support Drape & MF Large (use with CSM-2634)

Position over CSM-2634 to
provide safe and comfortable
support to the patient,
reducing the risk of pressure-
related injury.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



Medical Device



UK Conformity
Assessed



Do Not Reuse



CE Mark



Latex Free



Non Sterile



LOT Number:

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

EC REP

Product Code: LSD-350-L-03 Rev.3 02/24




Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.

The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin

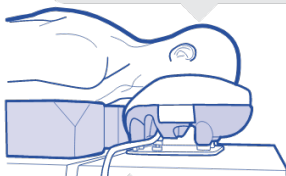


Neck

- + Chin and throat not touching the chest support
- + Natural, neutral neckline

Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breast positioned medically with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

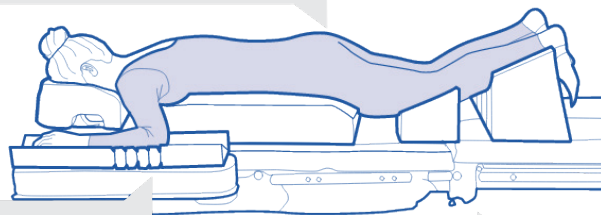

The arms

- + Arms forward and not abducted beyond 90° or arms secure by the patient's side
- + No pressure on the axillary nerve
- + No body weight supported on the arms

Legs and feet

- + No pressure on the thighs
- + Ankles raised

To see our full range of Care Surgical Medical Products, visit www.care-surgical.com



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



Medical Device



UK Conformity
Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 123108

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland