




CARE SURGICAL

Product Code: CSM2725

CS Jackson Style Chest Support

⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

Patient Supports for use on dual-rail Jackson style operating tables. Refer to the label on the support to ensure it is placed in the correct position on the table.

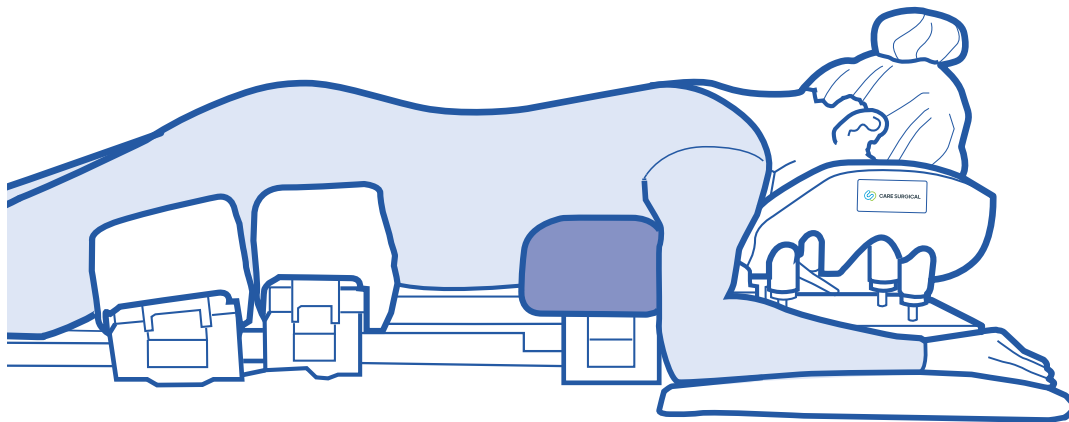

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland

EU REP

MD

②

⊘ LATEX

⚠ NON
STERILE

UK
CA

CE

250kg

Maximum
Patient Weight




SAFE DISPOSAL: customers should adhere to any local laws
and regulations to the safe disposal of medical devices.




DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury,
please contact Care Surgical.




Ref: CSM2725 - IFU (Rev.02) 03/2025

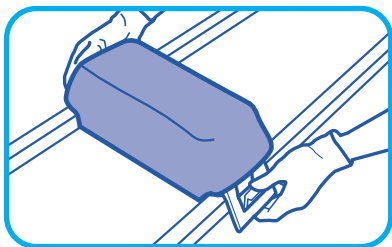

- 1** Place the support on to the frame so that the brackets are over the bars.

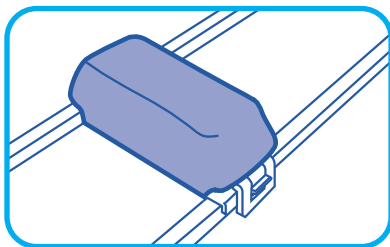

- 2** Snap the locking clip shut to secure the support into position.

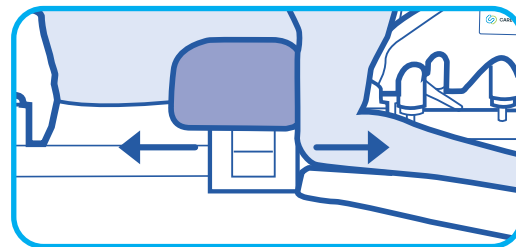

- 3** To adjust the position of the supports, release the clip(s), and slide the support(s) to the correct position, then snap lock the clip(s).

Ensure the iliac crest is in the centre of the hip pads.
Do not exceed the weight capacity of the operating table.


Cleaning instructions:



Clean using standard hospital wipes



Do not immerse in water

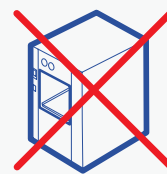

Do not autoclave



UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Maximum Patient Weight



SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.
DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.



EU REP