




# CARE SURGICAL


## CS Ultrafin Compatible Replacement Pads

CS Ultrafin Compatible Replacement Pads; use to support the patient's limb safely and comfortably whilst in the Ultrafin Stirrup

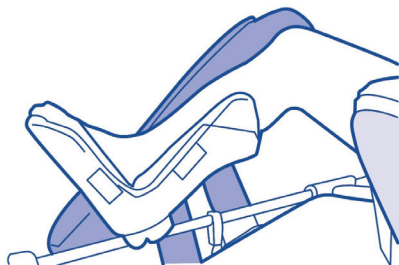


1

Position the patients leg and foot in the stirrup.

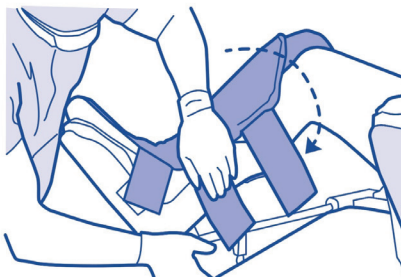

2

Ensuring the patient's leg is positioned correctly, take the stirrup pad straps and wrap the pad over and around the leg, enclosing the patients limb.

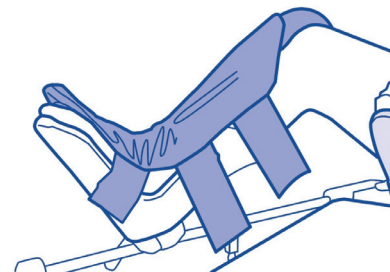

3

Ensure the pad is attached securely utilising the Velcro® straps.



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

**www.care-surgical.com**

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



LOT Number:

Product Code: CSM-2741

Rev 3

02/24

EC REP



## Pad cleaning and disinfection

### Cleaning

Clean the pad after each use. Clean using neutral warm soapy water. Ensure the pad is dry before storage. **DO NOT IMMERSE IN WATER**




Clean using standard hospital wipes



Do not immerse in water

### Disinfection

Disinfect using quaternary solution following the manufacturers recommendation for achieving low level disinfection. Wipe dry with a clean cloth.

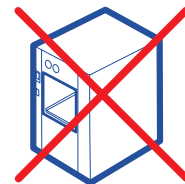

Do not autoclave



#### CAUTION:

Do not use cleaning solution at full strength



Do not apply tape to this pad



Do not use Bleach



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



5 060624 122149

[www.care-surgical.com](http://www.care-surgical.com)



Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland