



CARE SURGICAL



Product Code: CSM2907

CS Universal Half-Roll

⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

Reusable half-roll foam support designed for safe and comfortable positioning of the patient to reduce risk of pressure-related injury e.g. to support above the hollow of the patient's knee when in supine position.

*Never position directly in the hollow of the knee to prevent damage.

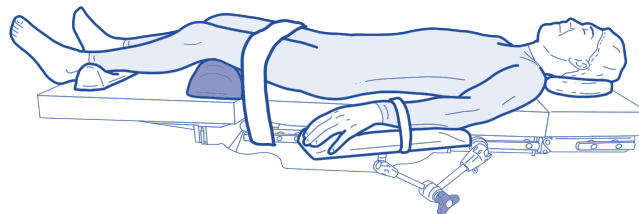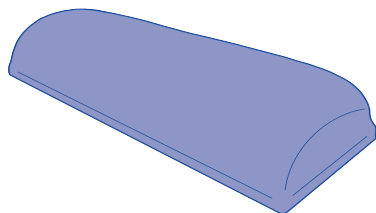

Cleaning instructions:



Clean using standard hospital wipes



Do not immerse in water



Do not autoclave



UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



Maximum Patient Weight



www.care-surgical.com



Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

⚠ **SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.
⚠ **DO NOT USE** products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.



5 060624 129070

Ref: CSM2907 - IFU (Rev.06) 04/2025