




# CARE SURGICAL

Product Code: CSM2699, CSM2918, CSM2919

## CS Easy Prone™ - Chest & Pelvis Set (150mm/170mm/200mm)

**⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.**

**Thorax and Pelvic Support for use on operating tables to position the patient's thorax and pelvic in prone position.**

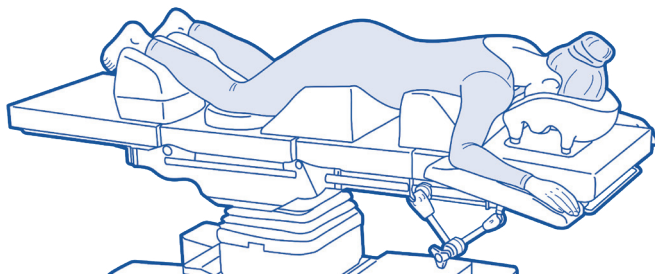

CSM2699:



CSM2918:



CSM2919:



### Cleaning instructions



Clean with standard hospital wipes

Do not immerse in water



Do not autoclave

**Refer to the label on the support to ensure it is placed in the correct position on the table**

- + Before repositioning, make sure that the supports are already in the correct position on the operating table. Thorax support at the level of the patient's nipple line, pelvic support at the level of the iliac crest.
- + After repositioning, the larynx, axilla, abdomen and genitals should be positioned pressure-free.
- + Suitable for male patients, as well as female patients with small to normal sized breasts.
- + Not suitable for female patients with large breasts or implants.



UK: Care Surgical Ltd  
Unit 6 Ringtail Road,  
Burscough,  
L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

**www.care-surgical.com**

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Luacan Co. Dublin, K78 X5P8 Ireland



**SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.



**DO NOT USE** products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

✓/X



CSM2699:



✓/X



CSM2918:



✓/X



CSM2919:



EU REP