CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2649

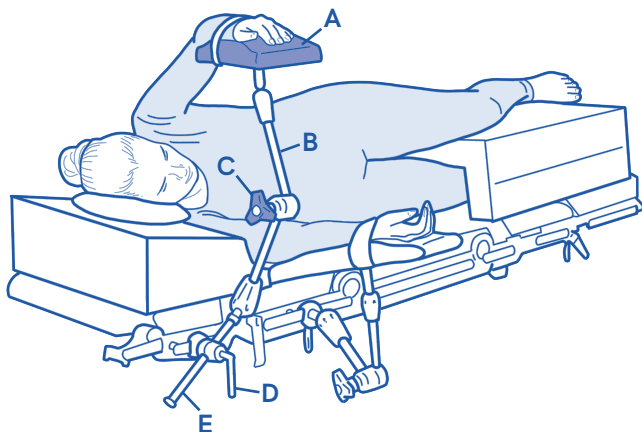
CS Universal Arm Support - Extended

CARE SURGICAL

Product Code: CSM2649

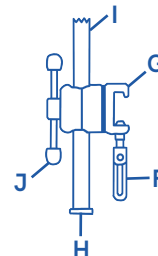
CS Universal Arm Support™ - Extended

Only to be used by trained clinicians. Ensure product is secure before positioning patient.

CS Universal Arm Support™ - Extended; used to position the patients arm in the prone, supine or lateral position.

A = Pad
B = Articulated Arm
C = Central Handle
D = Rail Clamp
E = Column

The Rail Clamp (D):
F = Fastening Screw
G = Upper Hook
H = Stop Lid
I(E) = Column
J = Fastening Handle

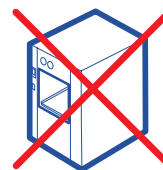
Cleaning instructions:

Clean using standard hospital wipes

Do not immerse in water

Do not autoclave

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.

DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury,
please contact Care Surgical.

5 060624 126499

EU REP

CARE SURGICAL

CS Universal Arm Support™ - Extended

Attachment to the table:

1. Attach the rail clamp (D) to the side of the surgical table and secure it by turning the fastening screw (F) from below clockwise
2. Pay attention during attachment that the upper hook (G) grabs behind the rail. Check if the rail clamp (D) is fixed securely and firmly on the side rail
3. Remove the stop lid (H) and insert the column (I) into the opening of the radial setting clamp.
4. Fix it with the fastening handle (J)

Mount the stop lid

5. Check if the column is fixed securely and firmly into the rail clamp. The column can be adjusted for height and inclination.

Holding-Releasing-Positioning-Fixing-Controlling:

6. Hold the articulated arm with one hand in the anterior segment (A) and manipulate the central handle (C) with the other hand.
7. To release turn the central handle (C) counterclockwise as far as necessary.
8. Move the articulated arm into the desired position
9. To secure, turn the central handle (C) clockwise
10. Controlling: Check that the articulated arm is firmly tightened and functioning properly.

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

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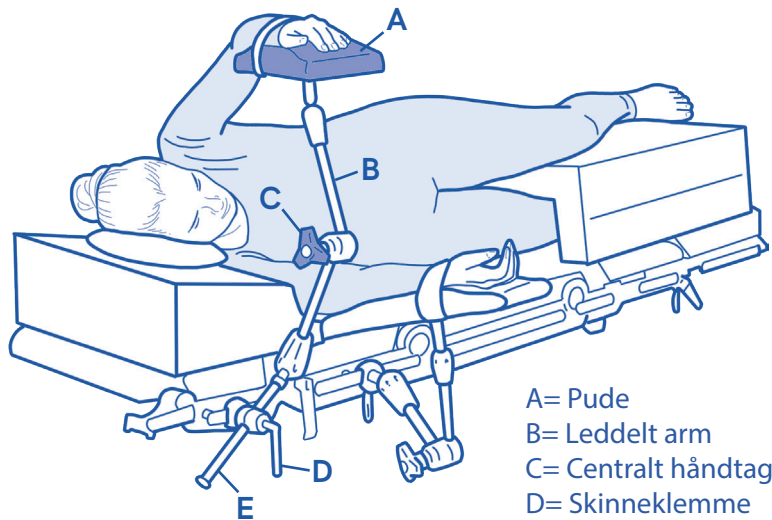
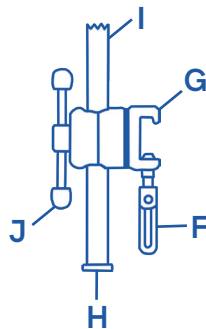
Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

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