CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CS Easy Prone

CSM2699 CS Easy Prone Chest and Pelvis Set - 150mm

CSM2918 CS Easy Prone Chest and Pelvis Set - 170mm

CSM2919 CS Easy Prone Chest and Pelvis Set - 200mm


CARE SURGICAL

Product Code: CSM2699, CSM2918, CSM2919

CS Easy Prone™ - Chest & Pelvis Set (150mm/170mm/200mm)

⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

Thorax and Pelvic Support for use on operating tables to position the patient's thorax and pelvic in prone position.

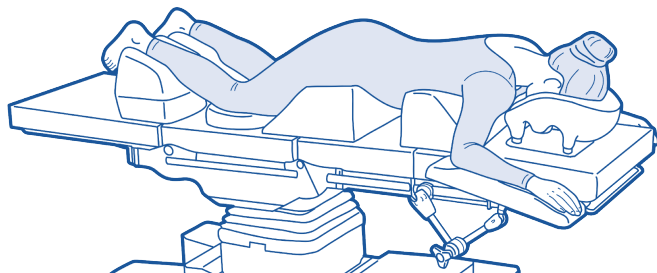CSM2699:

CSM2918:

CSM2919:


Cleaning instructions

Clean with standard hospital wipes

Do not immerse in water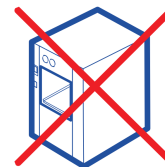

Do not autoclave

Refer to the label on the support to ensure it is placed in the correct position on the table

- + Before repositioning, make sure that the supports are already in the correct position on the operating table. Thorax support at the level of the patient's nipple line, pelvic support at the level of the iliac crest.
- + After repositioning, the larynx, axilla, abdomen and genitals should be positioned pressure-free.
- + Suitable for male patients, as well as female patients with small to normal sized breasts.
- + Not suitable for female patients with large breasts or implants.

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.

DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.

CSM2699:

5 060624 126994

CSM2918:

5 060624 129186

CSM2919:

5 060624 129193

EU REP

CARE SURGICAL

Tuotekoodi: CSM2699, CSM2918, CSM2919

CS Easy Prone™ - Chest & Pelvis Set (150mm/170mm/200mm)

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