



Technical Data Sheet

CSM-2469 CS Universal Arm Support - Extended



Description

- Supports the patient's arm when undergoing surgical procedures
- Reusable
- Attaches to most standard and specialist operating tables to support patients' arms in various positions during surgical procedures (clamp for din rails included)
- Especially suitable for positioning the upper arm in the lateral position
- Supports maximum weight of 6kg
- Product weight 5.4kg

Application Area

- OR
- Anaesthesia

Measurements

- Length 460mm/18.11"
- Width 150mm/5.91"
- Height 910mm/35.83"
- Max Height (Rail to Top) 870mm/34.25"

Material Composition

- Polyurethane foam
- Latex & phthalate free
- stainless steel

Cleaning

- Clean using alcohol-based hospital wipes

Order Information

Product Code	Product Name	Qty
CSM-2649	CS Universal Arm Support - Extended	1 Unit

Packaging Information

- Each arm board is packaged in a recyclable polyethylene bag
- 1 unit is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

- Incorrect positioning of the patient on this product can cause:
 - Pressure marking or ulcers
 - Neurological damage, which can cause paraplegia;
 - Venous pressure, which can lead to ischaemia
- Ensure clamps are properly tightened before use to avoid unwanted movement of the arm board.
- Supports maximum weight of 6 kg
- Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA: Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624126499

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
Marlborough
MA 01752
United States
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.02 11/2024