



# Technical Data Sheet

## CSM2649 CS Universal Arm Support - Extended™



### Description

- Supports the patient's arm when undergoing surgical procedures
- Reusable
- Attaches to most standard and specialist operating tables to support patients' arms in various positions during surgical procedures (clamp for din rails included)
- Especially suitable for positioning the upper arm in the lateral position
- Supports maximum weight of 6kg
- Product weight 5.4kg

### Application Area

- OR
- Anaesthesia

### Measurements

- Length: 460mm/18.11"
- Width: 150mm/5.91"
- Height: 910mm/35.83"
- Max Height: (Rail to Top) 870mm/34.25"

### Material Composition

- Polyurethane foam
- Latex & phthalate free
- stainless steel

### Cleaning

- Clean using alcohol-based hospital wipes

### Order Information

Product Code	Product Name	Qty
CSM2649	CS Universal Arm Support - Extended™	1 Unit

### Packaging Information

- Each arm board is packaged in a recyclable polyethene bag
- 1 unit is packaged into a cardboard box for sale

### Storage Instructions

- First in first out
- Store in a cool dry place
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

### Contraindications

- Incorrect positioning of the patient on this product can cause:
  - Pressure marking or ulcers
  - Neurological damage, which can cause paraplegia;
  - Venous pressure, which can lead to ischaemia
- Ensure clamps are properly tightened before use to avoid unwanted movement of the arm board.
- Supports maximum weight of 6 kg
- Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

### Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/UKCA:Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624126499

### Labelling



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
United Kingdom  
Tel: +44 1704 336671

US: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson  
MA 01749, USA  
Tel: +1-866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark