





CARE SURGICAL

CS Universal Arm Support-Extended

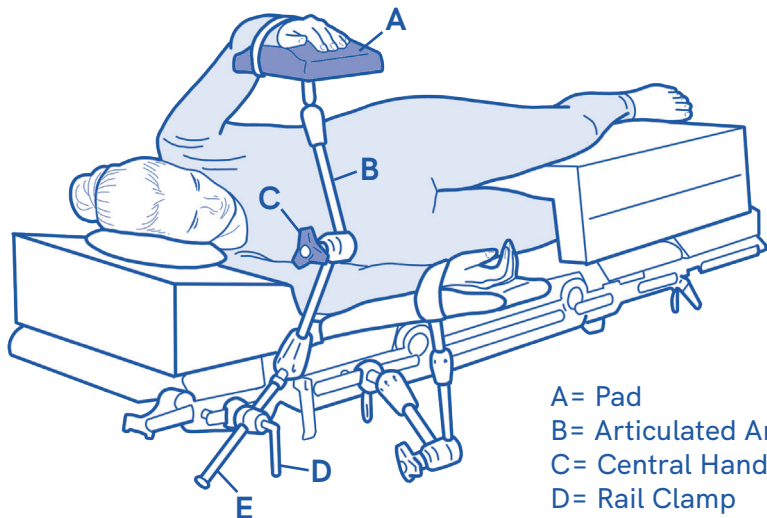

A = Pad
B = Articulated Arm
C = Central Handle
D = Rail Clamp
E = Column

CS Universal Arm Support-Extended; used to position the patients arm in the prone, supine or lateral position.

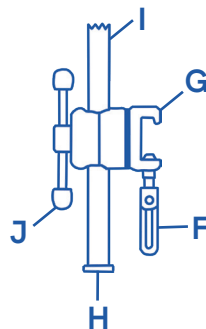

The Rail Clamp (D):
F = Fastening Screw
G = Upper Hook
H = Stop Lid
I(E) = Column
J = Fastening Handle



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



Medical Device



UK Conformity
Assessed



Reuse



CE Mark



Latex Free



Non-Sterile



LOT Number:

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

EC REP

Product Code: CSM-2649 Rev.2 02/24



CARE SURGICAL

CS Universal Arm Support- Extended

Attachment to the table:

1. Attach the rail clamp (D) to the side of the surgical table and secure it by turning the fastening screw (F) from below clockwise
2. Pay attention during attachment that the upper hook (G) grabs behind the rail. Check if the rail clamp (D) is fixed securely and firmly on the side rail
3. Remove the stop lid (H) and insert the column (I) into the opening of the radial setting clamp.
4. Fix it with the fastening handle (J)

Mount the stop lid

5. Check if the column is fixed securely and firmly into the rail clamp. The column can be adjusted for height and inclination.

Holding-Releasing-Positioning-Fixing-Controlling:

6. Hold the articulated arm with one hand in the anterior segment (A) and manipulate the central handle (C) with the other hand.
7. To release turn the central handle (C) counterclockwise as far as necessary.
8. Move the articulated arm into the desired position
9. To secure, turn the central handle (C) clockwise
10. Controlling: Check that the articulated arm is firmly tightened and functioning properly.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



Medical Device



UK Conformity
Assessed



Reuse



CE Mark



Latex Free



Non-Sterile



5 060624 126499

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

EC REP

Product Code: CSM-2649 Rev. 2 02/24