




CARE SURGICAL

# CS Easy Prone - 150mm - Chest & Pelvis Set

Thorax and Pelvic Support for use on operating tables to position the patient's thorax and pelvic in prone position.

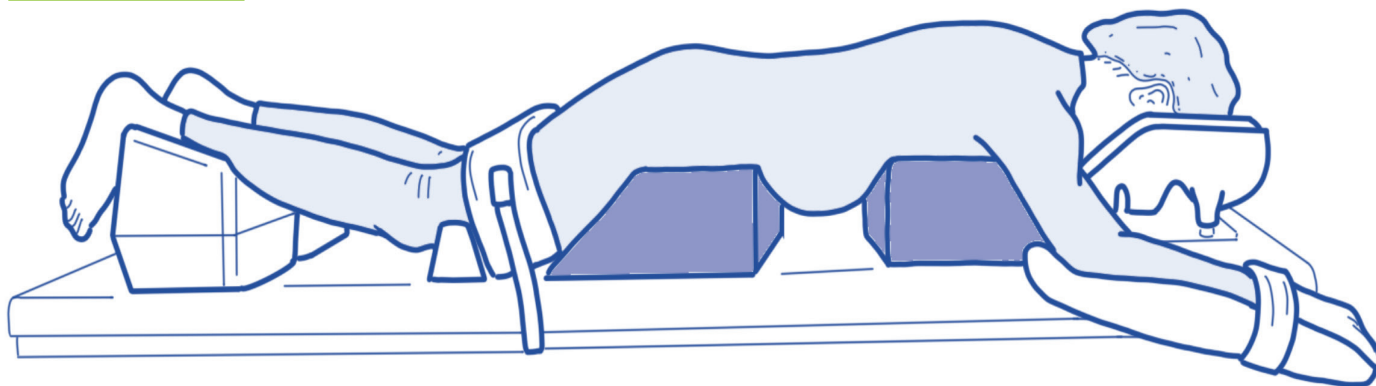

Refer to the label on the support to ensure it is placed in the correct position on the table



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

**www.care-surgical.com**

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity  
Assessed



Reuse



CE Mark



Latex Free



Non Sterile



LOT Number:

EC REP

Product Code: CSM-2699 Rev.3 02/24



# CARE SURGICAL

## CS Easy Prone - 150mm - Chest & Pelvis Set

- + Before repositioning, make sure that the supports are already in the correct position on the operating table. Thorax support at the level of the patient's nipple line, pelvic support at the level of the iliac crest.
- + After repositioning, the larynx, axilla, abdomen and genitals should be positioned pressure-free.
- + Suitable for male patients, as well as female patients with small to normal sized breasts.
- + Not suitable for female patients with large breasts or implants.

### Cleaning instructions



Clean with standard  
hospital wipes



Do not immerse in water



Do not autoclave



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

**www.care-surgical.com**

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity  
Assessed



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 126994

EC REP

Product Code: CSM-2699 Rev.3 02/24