




CARE SURGICAL

Product Code: CSM2926

CS Easy Prone Comfort Covers

⚠ Only to be used by trained clinicians.

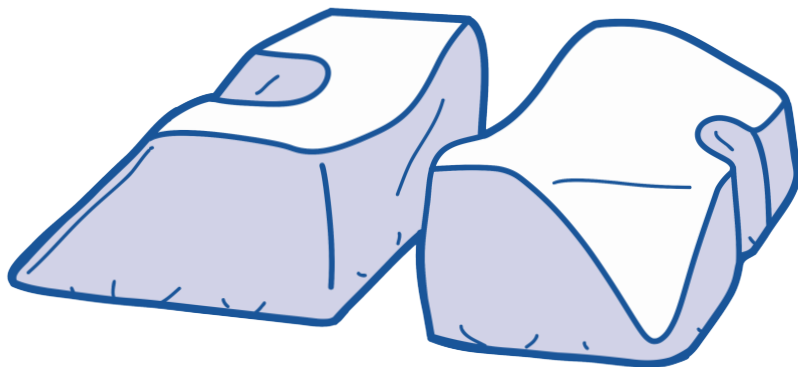

CS Easy Prone Comfort Covers -1 Thorax- & 1 Pelvis Cover

Enhances infection control and protects the pads from damage. Reduces the chance of pressure related injury.

Compatible with the CS Easy Prone CSM-2699,-2918, -2919



UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



LOT

⚠ **DO NOT REUSE!** if reused injury to patient could occur due to issues with infection control and deterioration of the product.
⚠ **SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.
⚠ **DO NOT USE** products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

EC REP

Ref: CSM2926 - IFU (Rev.03) 04/2025




Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.

The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin

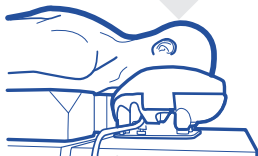


Neck

- + Chin and throat not touching the chest support
- + Natural, neutral neckline

Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breast positioned medically with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

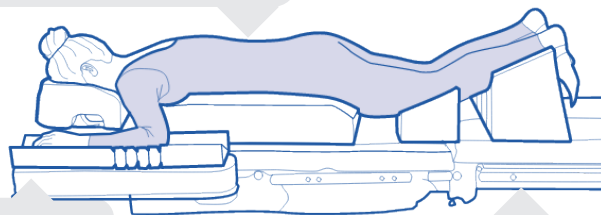

The arms

- + Arms forward and not abducted beyond 90° or arms secure by the patient's side
- + No pressure on the axillary nerve
- + No body weight supported on the arms

Legs and feet

- + No pressure on the thighs
- + Ankles raised

To see our full range of Care Surgical Medical Products, visit www.care-surgical.com



UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

