





CARE SURGICAL

Product Code: CSM3048


CS Mayfield/Doro Adaptors

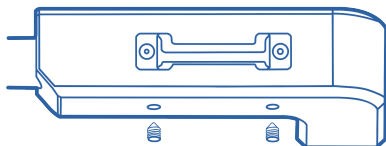


- 1 Remove the grub screws from underside of the Frame (head end).

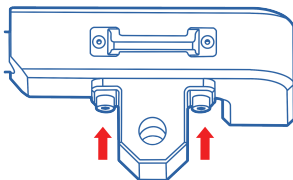


- 4 Bolt the adaptor into the frame (remove wing nut if needed).

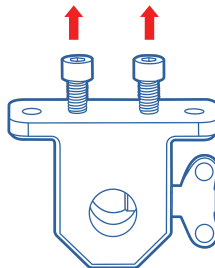


- 2 Using 6mm allen key, remove bolts from adaptor.

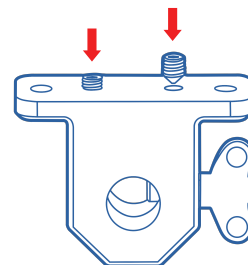

- 3 Store grub screws in adaptor. Ensure grub screws are not protruding.

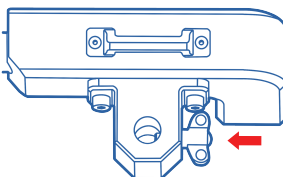

- 5 Reinsert wing nut.

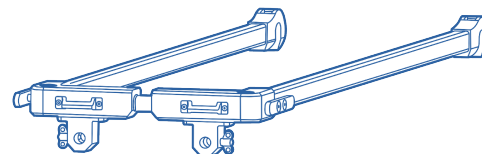

⚠ Only to be used by trained clinicians.



UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.



DO NOT USE products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

EU REP