



# Technical Data Sheet

## CSM3031 CS Universal Spine Extension™ Set



### Description

- Dual rail extension frame intended to be attached to an operating table, to which specialist supports can be attached to position patients for surgery (e.g. spinal surgery).
- Compatible with sectional pressure management pads and other support attachments (hip/thigh supports).
- Clean using non-alcohol hospital wipes
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)
- Latex and phthalate free
- Instructions for use are packaged with the product

### Application Area

- OR
- Anaesthesia

### Measurements

- Max Dims - L 1595mm | W 406mm | H 960mm
- Min Dims- L1595mm | W 406mm | H 560mm

### Material Composition

- Stainless steel
- Carbon Fibre
- Aluminium
- Polypropylene
- Rubber
- Impact resistant plastic

### Cleaning

- Clean with standard hospital wipes

### Order Information

Product Code	Product Name	Qty
CSM3031	CS Universal Spine Extension™ Set	1 Set

### Packaging Information

- Each unit is packaged in a recyclable polyethylene bag and bubblewrap
- 1 unit is packaged into a cardboard box for sale

### Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

### Contraindications

- Ensure the extension frame is securely attached to the table before use
- Check if the extension frame is damaged before and after each use. Do not use if damaged and contact supplier for advice
- Do not exceed the maximum weight mentioned on the IFU
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

### Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624126000

### Labelling



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
United Kingdom  
Tel: +44 1704 336671

US: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson  
MA 01749, USA  
Tel: +1-866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.01 01/2026



# Technical Data Sheet

## CSM2956 CS Universal Spine Extension™ - Powered Leg



### Description

- Electrical telescopic leg
- Moves respective to the surgical table
- used with CSM 2955 and CSM 2957
- Clean using non-alcohol hospital wipes
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)
- Latex and phthalate free
- Instructions for use are packaged with the product

### Application Area

- OR
- Anaesthesia

### Measurements

- Max Dims: L 960mm | W 460mm | H 406mm
- Min Dims: L 560mm | W 460mm | H 406mm

### Material Composition

- Stainless steel
- Polypropylene
- Rubber
- Aluminium
- Impact resistant plastic

### Cleaning

- Clean with standard hospital wipes

### Order Information

Product Code	Product Name	Qty
CSM2956	CS Universal Spine Extension™ - Powered Leg	1 Unit

### Packaging Information

- Each unit is packaged in a recyclable polyethylene bag and bubblewrap
- 1 unit is packaged into a cardboard box for sale

### Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

### Contraindications

- Ensure brakes are engaged before product is in use
- Check if the leg is damaged before and after each use. Do not use if damaged and contact supplier for advice
- Do not exceed the maximum weight mentioned on the IFU
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

### Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624126017

### Labelling



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
United Kingdom  
Tel: +44 1704 336671

US: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson  
MA 01749, USA  
Tel: +1-866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.01 01/2026



# Technical Data Sheet

## CSM2957 CS Universal Spine Extension™ - Frame



### Description

- Frame supports the patient and supports positioning equipment
- Connects to CSM 2955 and CSM 2956
- Clamp positioning equipment onto frame members
- Clean using non-alcohol hospital wipes
- Moves respective to the surgical table
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)
- Latex and phthalate free
- Instructions for use are packaged with the product

### Application Area

- OR
- Anaesthesia

### Measurements

- L 1242mm | W 502mm | H 76mm

### Material Composition

- Stainless steel
- Carbon Fibre
- Aluminium

### Cleaning

- Clean with standard hospital wipes

### Order Information

Product Code	Product Name	Qty
CSM2957	CS Universal Spine Extension™ - Frame	1 Unit

### Packaging Information

- Each unit is packaged in a recyclable polyethylene bag and bubblewrap
- 1 unit is packaged into a cardboard box for sale

### Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

### Contraindications

- Ensure the frame is securely attached to the table adaptor before use
- Check if the frame is damaged before and after each use. Do not use if damaged and contact supplier for advice
- Do not exceed the maximum weight mentioned on the IFU
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

### Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624125997

### Labelling



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
United Kingdom  
Tel: +44 1704 336671

US: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson  
MA 01749, USA  
Tel: +1-866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.01 01/2026



# Technical Data Sheet

## CSM3030 CS Universal Spine Extension™ - Table Adaptor US



### Order Information

Product Code	Product Name	Qty
CSM3030	CS Universal Spine Extension™ - Table Adaptor US	1 Unit

### Description

- Adaptable rail clamp that supports different width surgical tables
- Clamp onto surgical table rails. To be used with CSM2956 and CSM2957. CSM2957 (frame) attaches to CSM3030 (adaptor).
- For use on compatible surgical tables in surgical environments and ICUs
- Clean using non-alcohol hospital wipes
- Moves respective to the surgical table
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)
- Latex and phthalate free
- Instructions for use are packaged with the product

### Application Area

- OR
- Anaesthesia

### Measurements

- Max Dims: L 226mm | W 540.5/740.5mm | H 170mm

### Material Composition

- Stainless steel
- Carbon Fibre
- Aluminium

### Cleaning

- Clean with standard hospital wipes

### Packaging Information

- Each unit is packaged in a recyclable polyethylene bag and bubblewrap
- 1 unit is packaged into a cardboard box for sale

### Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

### Contraindications

- Ensure the adaptor is securely attached to the table before use
- Check the adaptor for damage before and after each use. Do not use if damaged Contact supplier for advice
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

### Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624129551

### Labelling



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
United Kingdom  
Tel: +44 1704 336671

US: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson  
MA 01749, USA  
Tel: +1-866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.01 01/2026