




CARE SURGICAL CS FlexiFin™ Stirrup Pads (Pair)

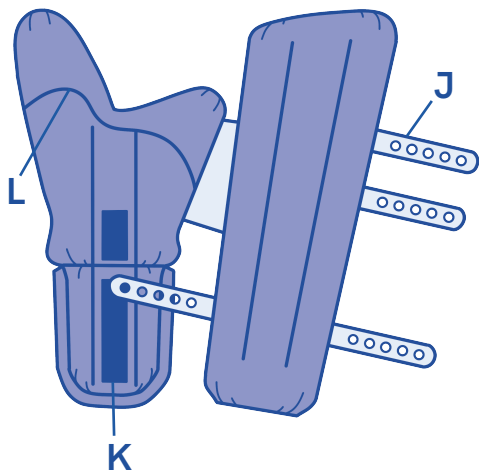

- 1 Position the patients leg and foot in the stirrup, ensure that the patient's heels are fully seated in the heel section of the boot
- 2 Close the stirrup boot pad with the straps by threading the appropriate hole over the hook provided. Ensure that the straps are fixed and the patient's leg is secured without too much tension on the strap or too much pressure on the back of the patient's foot.
- 3 In the event of heavy contamination, the pad can be removed for better cleaning. Make sure that all silicon fasteners (J) are loose and the hook & loop on the back of the pad (K) is carefully separated to prevent damage to the cover before removing the pad from the boot.
- 4 After cleaning (the pad should be completely dry), put the pad back in the boot so that the hook & loop straps on the back of the pad (K) and in the boot are on top of each other and pull the lip of the cover (L) over the edge of the boot fins.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

EC REP

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity Assessed



Reuse




CE Mark



Latex Free



Non Sterile

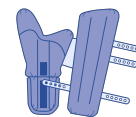

LOT

Blank boxes for LOT and other identification numbers.

Product Code: CSM-2996

Rev.1 02/24