



Technical Data Sheet

CSM2781 CS AlignPro™ Support



Description

- Prone positioning supports to comfortably position patients on a dual rail system with a high degree of adjustability
- Supports the patient's hips and chest in the prone position
- Open the clips to slide the supports laterally into the required position. Close the clip securely to ensure the support does not move during the procedure

Application Area

- OR

Measurements

- Length 416mm/16"
- Width 200mm/8"
- Height 265mm/10.4"

Material Composition

- Wipeable skin covering
- Polyurethane foam
- Transfer coated PU with anti-bacterial treatment on knitted polyester
- Nylon
- Stainless Steel
- Carbon Fibre

Cleaning

- Clean with standard hospital wipes

Order Information

Product Code	Product Name	Qty
CSM2781	CS AlignPro™ Support	1 Set

Packaging Information

- Each unit is packaged in a recyclable polyethene bag
- 1x system boxed individually. Include 2x base supports with full pads sets

Storage Instructions

- Store appropriately and do not place pressure/excessive weight on components to avoid damage and longevity of device
- Transport and store in dry conditions and out of direct light where possible
- Store appropriately and do not place pressure/excessive weight on components to avoid damage and longevity of device

Contraindications

If the supports are not fully attached, the supports could move during the procedure which could cause injury to the patient or clinician.

Incorrect positioning of the patient on this product could result in the patient experiencing:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia
- Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624126970

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.01 16/12/2025