




- 1 Remove the grub screws from underside of the Frame (head end).

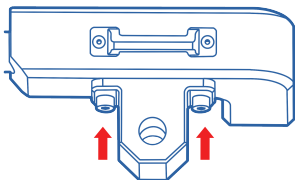


- 4 Bolt the adaptor into the frame (remove wing nut if needed).

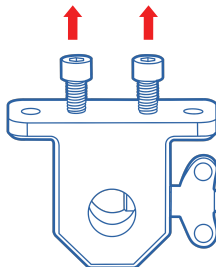


- 2 Using 6mm allen key, remove bolts from adaptor.

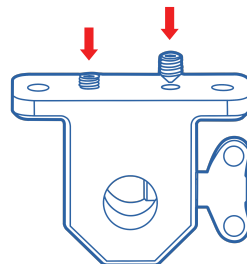


- 3 Store grub screws in adaptor. Ensure grub screws are not protruding.

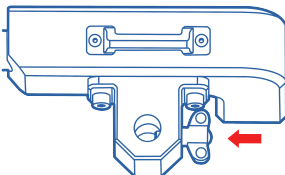

- 5 Reinsert wing nut.

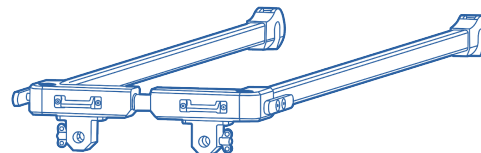

 Only to be used by trained clinicians.



UK: Care Surgical Ltd  
Unit 6 Ringtail Road,  
Burscough,  
L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



**SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.



**DO NOT USE** products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

